


NUT MUST BE GLUED WITH H900019

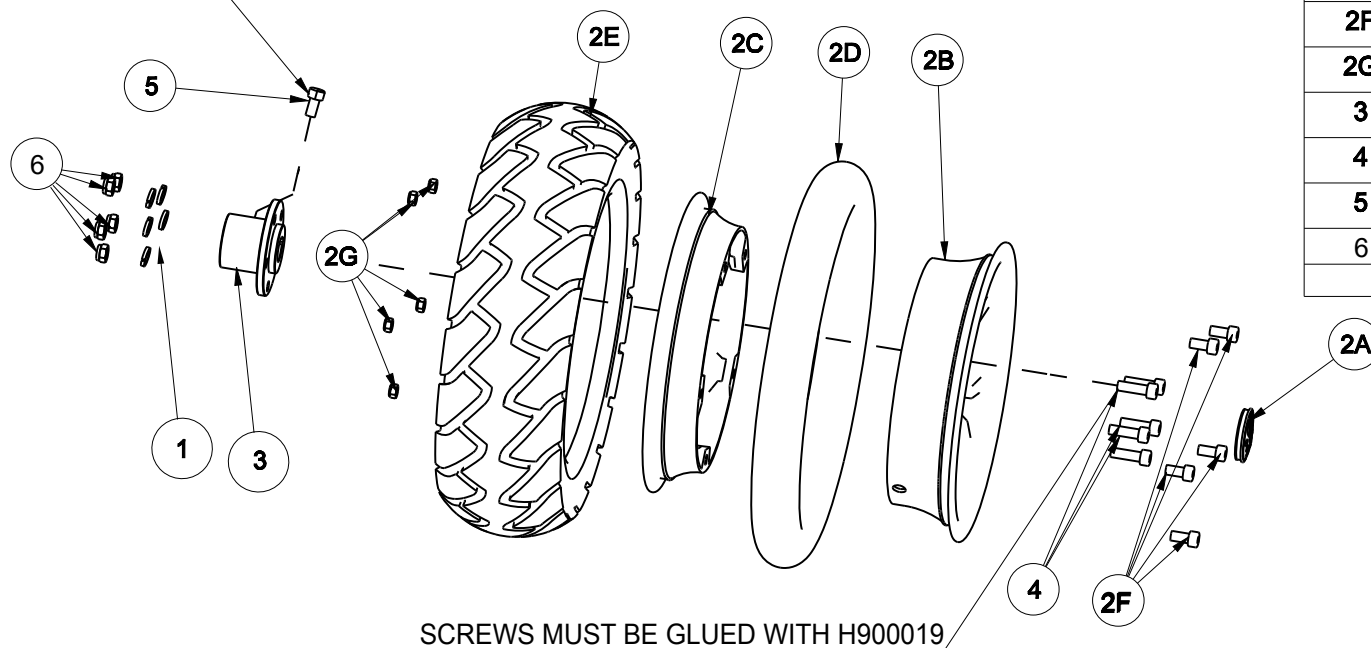

SCREWS MUST BE GLUED WITH H900019

Material			
No.	Plan	Article	Number
1	—	1001570	5
2	—	3990582	1
2A	X005032	1908252	1
2B	X005029	1772404	1
2C	X005030	1772504	1
2D	—	1006903	1
2E	—	3991196	1
2F	DIN912 M8X20	3991197	5
2G	—	3991743	5
3	XC00187	1C26800	1
4	DIN912 M8X25	3991096	5
5	DIN916 M8X30	1001443	1
6	---	3991743	5

3990523

EC00107/G

SV CARPO2
BACKWHEEL SET RIGHT

Scale: 1:5	Drafted	Verified	Confirmed
Date	08/10/2012	08/10/2018	08/10/2018
Init.	WW	Mars	TG

Plan No.
EC00107/G

