



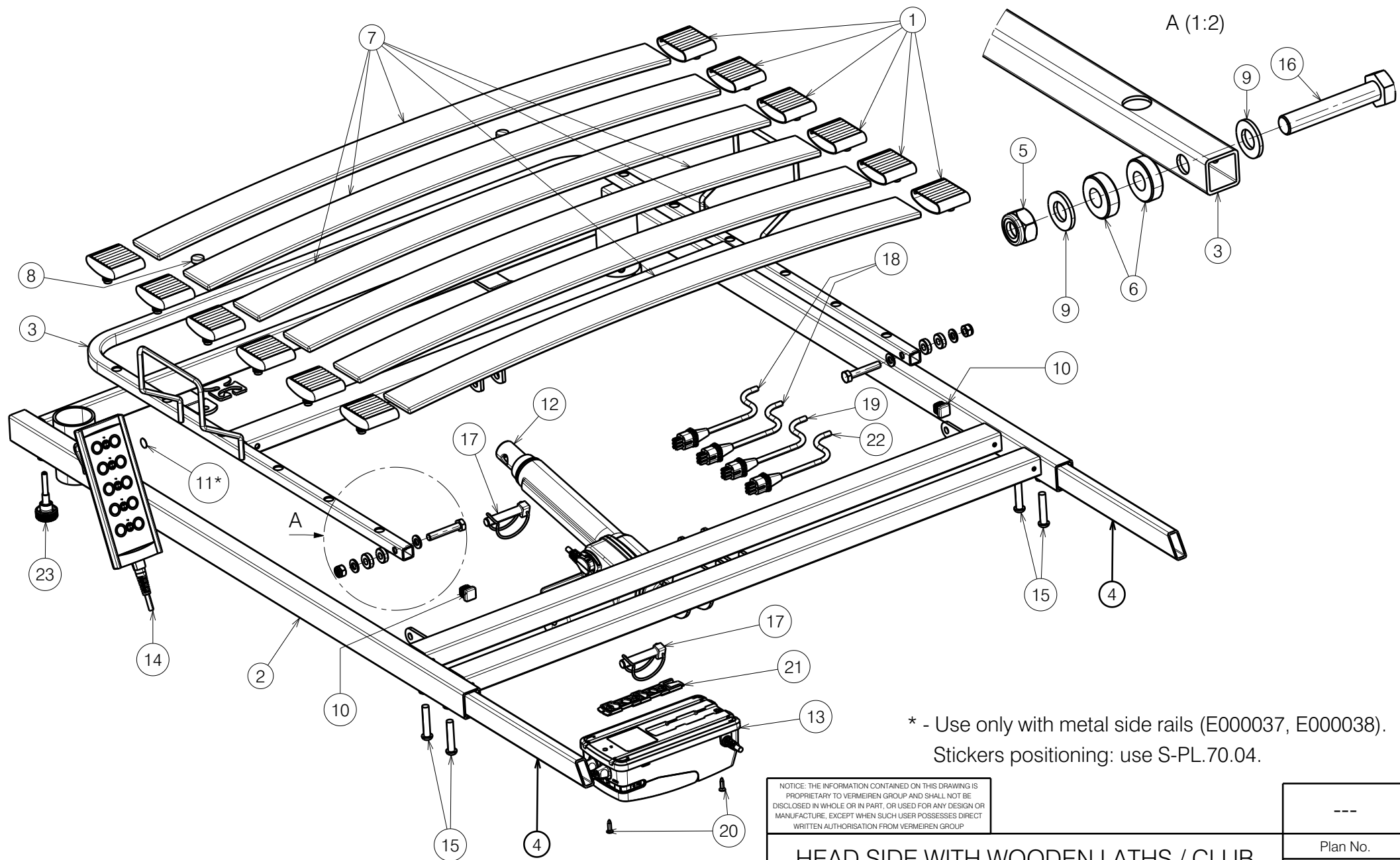

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

HEAD SIDE WITH WOODEN LATHS / CLUB 1/2				Plan No. E00044910/11.1
Scale: 1:6	Drafted	Verified	Confirmed	
Date	2022-10-27	07/07/2025	07/07/2025	
Initials	OY	PM	AZ	

No.	Plan	Article	Qty
1	P00033167	2995472	12
2	P00033418 DP	2650237	1
3	P00033422	2650137	1
4	YP08733	2626300	2
5	X001016	1001682	2
6	X004967	1908262	4
7	XP01421	2993116	6
8	---	1001637	2
9	D8,4 DIN125	1003199	4
10	Cap 15x15	1005899	2
11*	Arrow sticker	1903223	2
12	Motor LA24	1918548	1
13	Control Box	1918550	1
14	Remote Control	1919111	1
15	M8x40 ISO 7380	1008739	4
16	M8x45 DIN931	1005901	2
17	Pen D10x50	1998885	2
18	Panel Cable	1912902	2
19	Headrest Cable	1919185	1
20	D3,9x16 DIN7982	1005767	2
21	Control Box Holder	1919522	1
22	Footrest Cable	1919533	1
23	P00058901	2997834	2

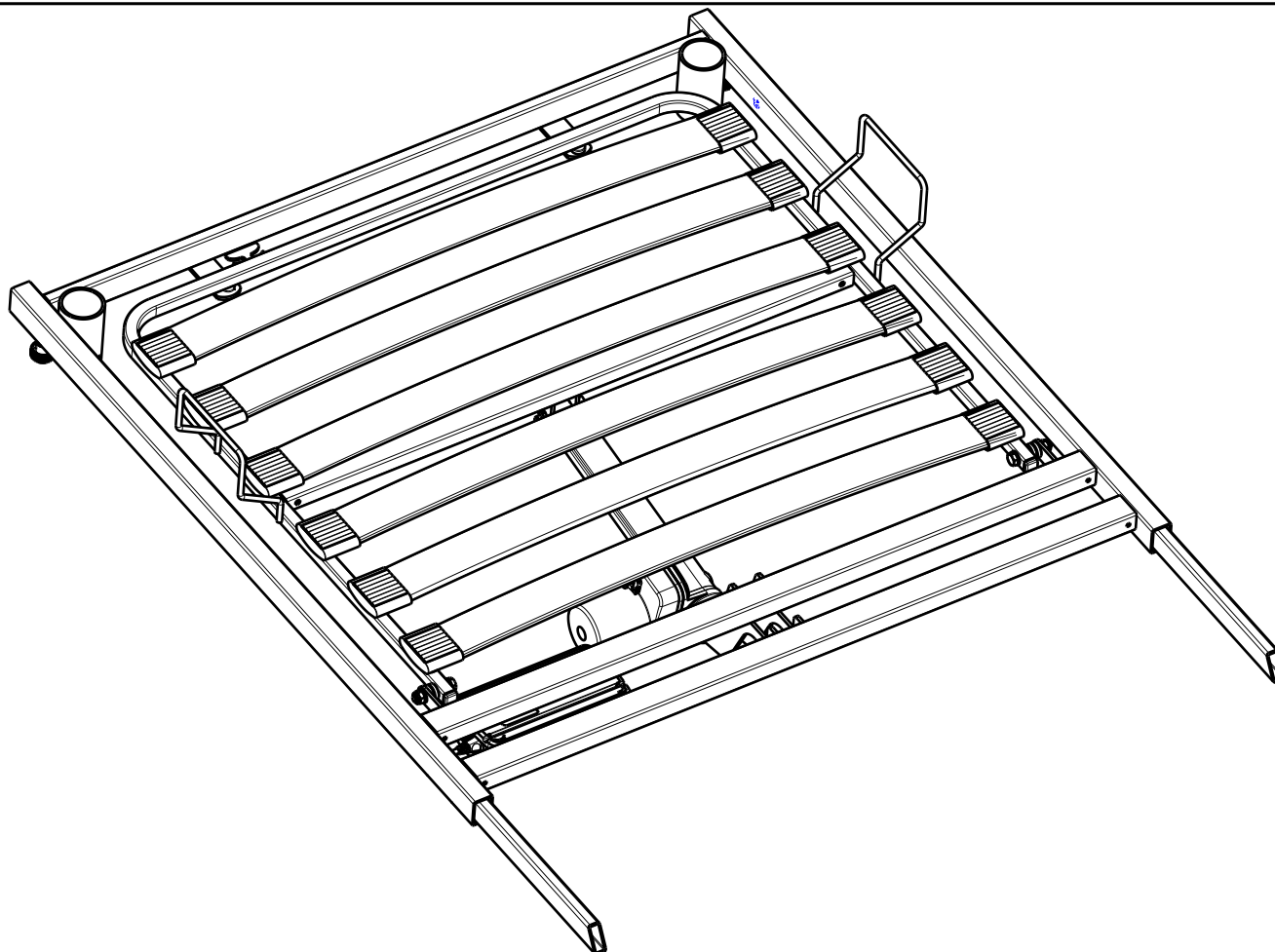

* - Use only with metal side rails (E000037, E000038).
Stickers positioning: use S-PL.70.04.

HEAD SIDE WITH WOODEN LATHS / CLUB ^{2/2}				Plan No.
				E00044910/11.1
Scale: 1:6	Drafted	Verified	Confirmed	
Date	2022-10-27	07/07/2025	07/07/2025	
Initials	OY	PM	AZ	