


EP00247 / G

<h2 style="margin: 0;">LOWER FRAME ILLICO</h2>				Plan No.
				1/2 EP00247/G
Scale: 1:8	Drafted	Verified	Confirmed	
Date	19/10/2017	24/08/2023	24/08/2023	
Initials	DB	AZ	TK	

Material			
No.	Plan	Article	Quantity
1	YP05847	27838	1
2	YP05862	27841	1
3	YP05872	27843	1
4	YP06097	2993740	6
5	YP06245	2795903	1
6	YP06445	27313	1
7	YP06446	27314	1
8	X001029	1375500	1
9	XP00871	2830300	1
10	XP01839	2993557	2
11	XP01931	2993584	12
12	Bearing D35xD17x10	1001409	4
13	Nut M8 DIN985	1001682	4
14	Nut M10 DIN985	1001683	6
15	Axle Ring DIN6799 D15	1003049	4
16	Washer M8 DIN125	1003199	9
17	M8x40 DIN7991	1008738	1
18	Nut M10	1009994	2
19	M8x60	1900809	1
20	Control Box Support	1913262	1
21	Wheel D100	1914317	4
22	M8x25	1990501	2
23	M10x90	1998812	4
24	M10x65	1998862	4
25	Pen D10x50	1998885	2
26	Cap 30x50x3	2993583	4
27	Pin D6x12 DIN1481	2993738	2
28	Motor LA40 8000N	1914183	1
29	Remote Control	1904296	1
30	Control Box CA40	1912898	1
31*	EPR Cable	1902971	1
32*	Motor Cable LA40	1912903	1
33*	Cable number 1	1995835	1

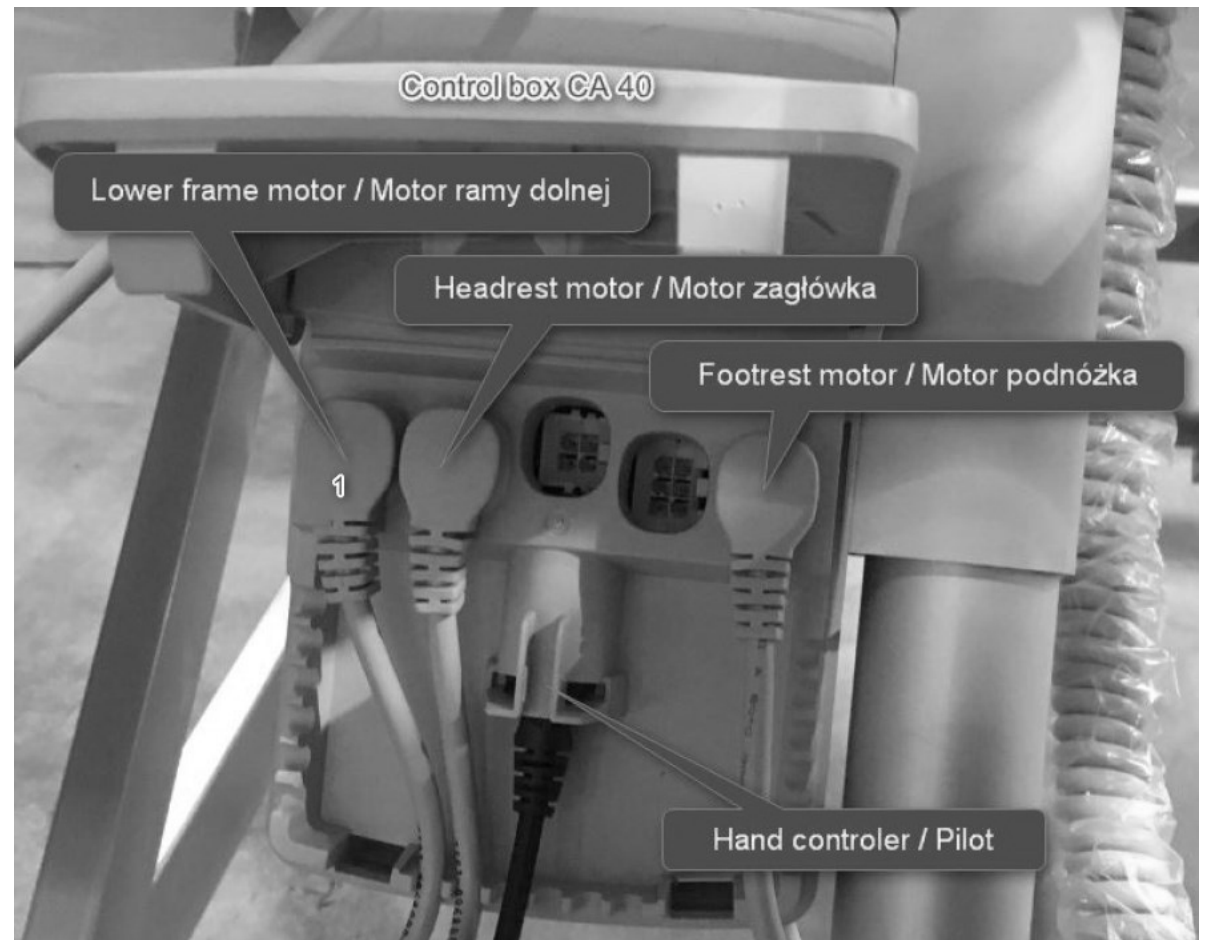

EP00247 / G

*- not depicted / nie ujęto

<p>LOWER FRAME ILLICO</p> <p style="text-align: right;">2/2</p>				Plan No.
				EP00247/G
Scale: 1:8	Drafted	Verified	Confirmed	
Date	19/10/2017	24/08/2023	24/08/2023	
Initials	DB	AZ	TK	